


LoFric Origo Pro – quick guide

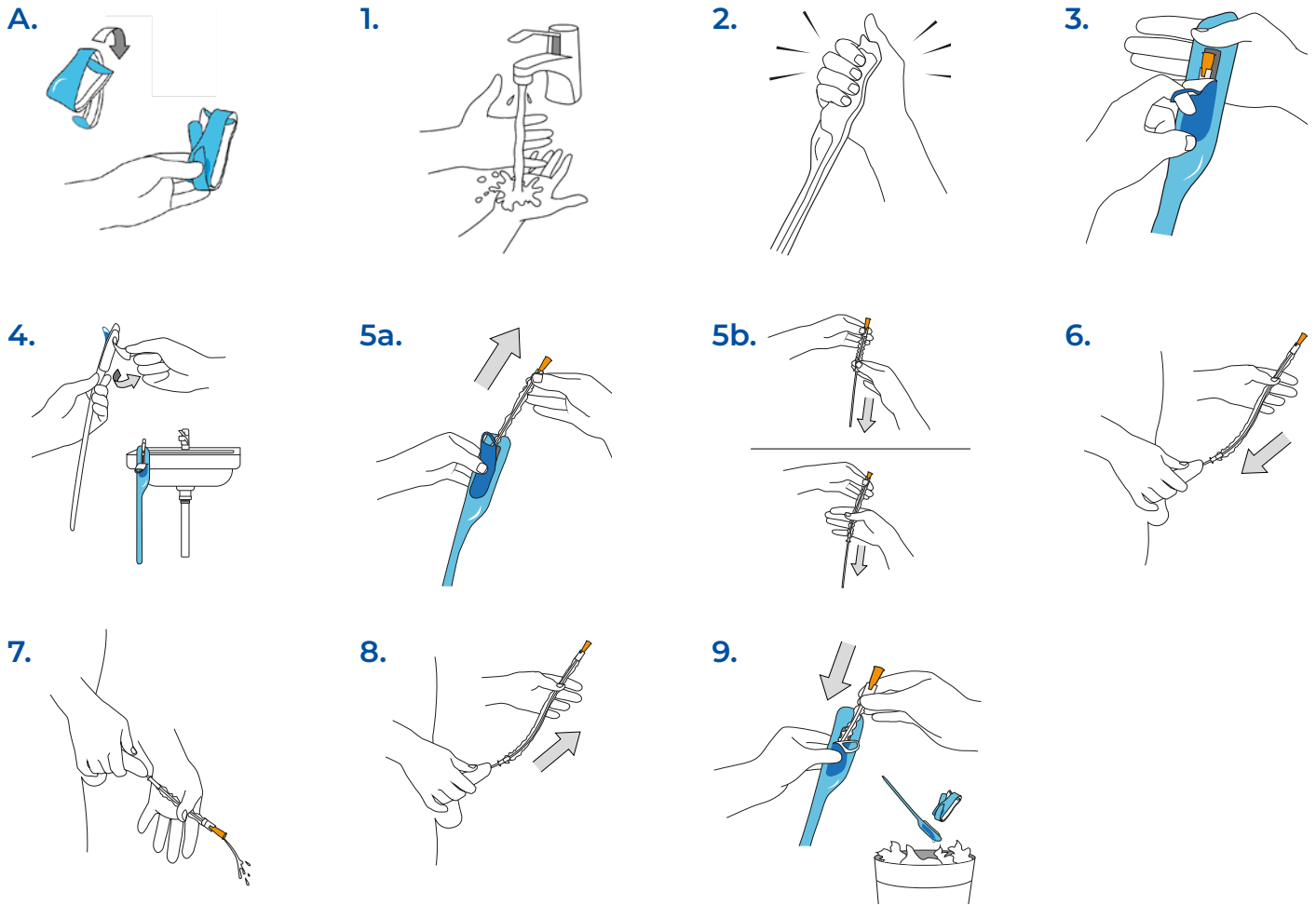

A. Tips for storing before use. Foldable to pocket size.
Use the slit to keep the catheter folded.

1. Wash your hands thoroughly with soap and water.
2. Squeeze to release the salt solution and the catheter is ready to use.
3. Use the loop to pull down and open.
4. Optional: Use the adhesive tab on the reverse side to attach the product to a dry, clean surface.
5. A. After opening, gently pinch and hold the sleeve grip to automatically pull down the protective sleeve when taking out the catheter. Use the protective sleeve to control insertion without having to touch the catheter tube.

B. OPTIONAL: Take out the catheter. Pull the sleeve grip down to cover the catheter tube. Use the protective sleeve to control insertion without having to touch the catheter tube.

6. Lift the penis towards the stomach to straighten the urethra. Slowly insert the catheter into the urethra until urine starts to flow, insert slightly more to ensure a steady stream.
7. Angle the penis down as urine begins to flow through the catheter.
8. Wait until the urine flow stops, to ensure complete bladder emptying. Slowly withdraw the catheter.
9. Put the catheter back in the package and dispose appropriately (local regulations may vary).

IMPORTANT INFORMATION

Use only the salt solution included to wet the catheter. Do not place the catheter on paper or cloth as these materials can absorb the wet coating.

Important information

LoFric[®] Origo[™] Pro, single use, hydrophilic urinary catheter, with salt solution for activation and multiple eyelets, designed to allow for complete bladder emptying without the need for repositioning.

For short and long term bladder management with intermittent urinary catheterization.

For adults, adolescents.

LoFric catheters are for prescription use only.

Ensure to follow the advice and training on the catheterization procedure given by your Healthcare Professional, before use.

Please refer to instructions for use included in the package, for important information and complete instructions.

Contact your prescriber if you experience difficulties.

At Wellspect we develop innovative continence care solutions that improve quality of life for people with bladder and bowel problems. We inspire our users to build self-confidence and independence as well as good health and well-being. We have been leading the industry for over 40 years with our product brands LoFric[®] and Navina[™]. We always aim to minimize the environmental impact of our products and passionately strive to become climate neutral. We work together with users and healthcare professionals to improve clinical outcome in a sustainable way, now and for the future.

Wellspect. A Real Difference.

For more information about our products and services, please visit [Wellspect.com](https://www.wellspect.com).

Join the conversation on LinkedIn, X, Facebook and Instagram.

[wellspect.com](https://www.wellspect.com)

Wellspect AB, Aminogatan 1, SE-431 21 Mölndal, Sweden. Phone: +46 (0)31 376 40 00.

LoFric



Manufacturer: Wellspect HealthCare, Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden.
Phone: +46 31 376 40 00. www.wellspect.com

CE 2797

R_x Only